

IMPROVING THE LIVES OF COUNTLESS CHILDREN AND ADULTS

Advanced Brain Technologies (ABT) is a neurotechnology company that develops and distributes interactive software and music-based programs for the improvement of memory, attention, listening, academic skills, sensory processing, brain health, peak performance, and more. ABT was founded in 1998 and is located in Ogden, Utah.

Our family of products includes an extensive line of interactive software and music programs including: The Listening Program®, BrainBuilder®, Sound Health®, Music for Babies™, Shelly's Interactive Computer Learning Series™, Spatial Surround® HD, ABT Digital Bone Conduction System™, and iListen®.

In addition, we offer professional training, certification, and continuing education opportunities to health, therapeutic, education, and music professionals as providers of The Listening Program®. Our international network includes providers implementing ABT products in home, school, clinic, hospital, and corporate settings on six continents.

ABT products are improving the lives of countless children and adults by training brain function at its core.

 **ADVANCED BRAIN TECHNOLOGIES**
5748 South Adams Avenue Parkway
Ogden, Utah 84405
www.advancedbrain.com



iListen® REGISTRATION

To receive service or warranty support, you must register your iListen and ABT Digital Bone Conduction System™ within 30 days of purchase. Go to www.thelisteningprogram.com and click on the “TLP Listener Resources” button located on the home page. Then select “iListen & ABT Bone Conduction System Registration” and complete the form.

WARRANTY

Advanced Brain Technologies (ABT) warrants the bone conduction amplifier and bone conduction headphones (Product) to be free from defects in material and workmanship for a period of one year. If a Product proves to be defective in material or workmanship during the warranty period, ABT will, at its sole option, repair or replace the Product with a similar Product. Replacement Product or parts may include remanufactured or refurbished parts or components. The replacement unit will be covered by the balance of the time remaining on the customer's original limited warranty. Replacement of lost files due to illegal file transfer attempts is specifically excluded under this warranty. The iPod® includes a one year warranty directly from Apple, Inc.

DISCLAIMER

The programs on iListen® have not been reviewed by the Food & Drug Administration (FDA). The programs on iListen are not intended to diagnose, treat, cure, or prevent any disease.

©2007-2008 Advanced Brain Technologies | All Rights Reserved
iListen® is a trademark owned by Advanced Brain Technologies. iPod® is a registered trademark of Apple, Inc. Packaging, system and product configuration, ©2007-2008 Advanced Brain Technologies, Ogden, Utah 84405. All rights reserved.

FOR ADDITIONAL INFORMATION OR SUPPORT, PLEASE CONTACT YOUR PROVIDER:

Visit us online at www.thelisteningprogram.com



ABT DIGITAL BONE CONDUCTION SYSTEM™
Quick Start Guide



CONNECTING YOUR UNIT



- 1 To connect iPod to Headphone Amplifier: Connect the enclosed 6” cable from the iPod Headphones port to the port marked IN on the Headphone Amplifier.
- 2 To connect Headphones to Headphone Amplifier: Connect the ALL BLACK connector on the Headphone cable to the port marked BC on the Headphone Amplifier. Then connect the connector with the BLUE RING on the Headphone cable to the port marked AC on the Headphone Amplifier.
- 3 To power Headphone Amplifier ON/OFF: Push the power button and the display will read “powering on”. To turn the amplifier off, push the power button and the display will read “powering off”.


PROTECT HEARING! To protect your hearing health, the maximum volume level of the iPod has been pre-set and locked by ABT. To change this setting, please contact your Provider.

PLAYING A PROGRAM

1. Unit is connected. DO NOT place Headphones on at this time.
2. Push MENU on the iPod Click Wheel. Make sure iPod is not on HOLD.
3. Select MUSIC. Choose your selection by lightly moving your thumb around the Click Wheel and clicking the Center Button on the iPod.
4. Select COVER FLOW and click the Center Button on the iPod.
5. Choose an ALBUM by lightly moving your thumb around the Click Wheel and clicking the Center Button on the iPod.
6. Now power on the Headphone Amplifier. Use the arrow keys on the front of the amplifier to select AC volume and turn volume to “0”.
7. Place Headphones on. Select a MODULE from the list and click Center Button on the iPod. Module will begin to play.
8. Adjust iPod volume to about 80%.
9. Use the arrow keys on the front of the amplifier to select and adjust the BC volume, AC volume and the Balance to the levels given to you by your Provider.
10. Be sure the volume is at a comfortable level. For any questions about volume settings, please contact your Provider.

WARNING! DO NOT attempt to connect the iPod to your computer or file loss may result. Neither Advanced Brain Technologies nor its Providers is responsible for loss of files. This unit is equipped with copy protection. Do not attempt to copy files from iListen®.

NOTE: To re-charge the iPod, please use the AC Power Recharger included in the iListen package. The Headphone Amplifier uses two (2) AA batteries.

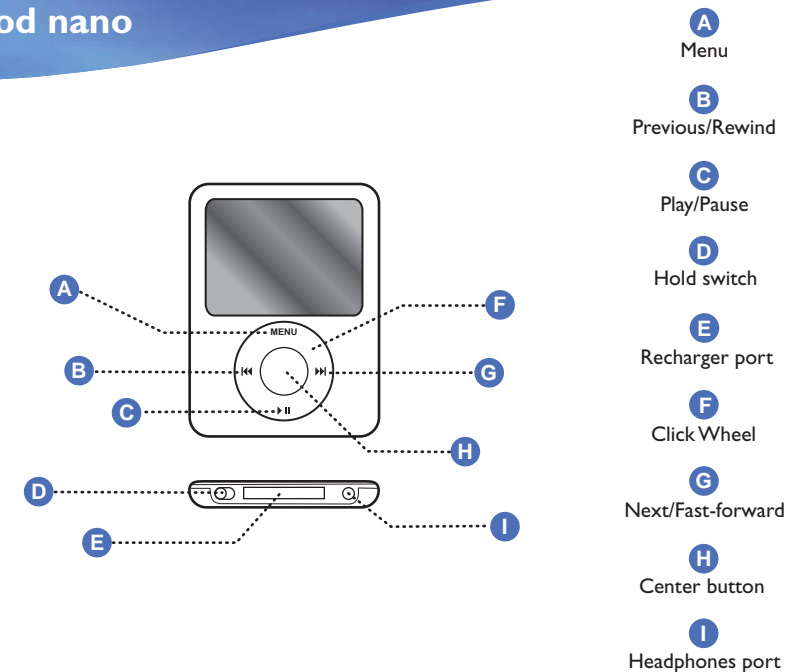

TO	DO THIS
Turn on iPod nano	Press any button.
Turn off iPod nano	Press and hold Play/Pause.
Turn on the backlight	Press any button or use the Click Wheel.
Go directly to the main menu	Press and hold Menu.
Change the volume	From the Now Playing screen, move your thumb around the Click Wheel.
Place iPod nano on HOLD (Disable iPod buttons to prevent accidental volume increases and control function mishaps while listening)	Slide the Hold switch to HOLD. An orange strip will appear when the iPod is on HOLD.
Re-charge iPod nano	Connect AC Power Recharger.
Reset, “unfreeze” an iPod nano that does not respond, won’t turn on, or remains blank	Check the Hold switch to make sure it’s not on HOLD. Check your battery to make sure it is charged. Slide the Hold switch to HOLD and back again. Then press the Menu and Center buttons at the same time until the Apple logo appears, about 6-10 seconds.
Service your iPod nano	Complete the online iPod request form found by selecting the Support tab at the top of apple.com. You can also take your iPod to an Apple retail store or an Apple Authorized Service Provider.